

M11259

Standard Folding Support Rail with Leg

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

Maximum User Weight



1. INTENDED USE

To provide a stable support to assist the user raise and lower themselves for toileting. The rail can be raised back to the wall when not in use or when transferring from the side. It is not intended for use in a shower.



2. SAFETY

WARNING: The maximum user weight must not be more than 100 kg (16 st).

This device should only be secured to a solid wall of brick or concrete construction and be installed by a suitably qualified person.

Test the rail after installation to ensure that it is securely fixed to the wall.

Periodically check the stability and rigidity of the product and, where applicable, that all screws are tightened and in good condition.

3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

4. ASSEMBLY/INSTALLATION

1. Ensure that when correctly located, the rail can be raised and lowered without interference from other fixtures and furniture and that transfer can be carried out safely.
2. Using the pin clip, adjust the leg length to the required position/rail height.
3. Position the Wall Plate against the wall at the height required (with leg square to and in contact with the floor) and mark the position of the 6 holes.
4. Drill 6 holes, as appropriate for the fixings/screws being used. Insert wall plugs and gently tap them home.
5. Place Wall plate against wall and locate suitable screws into the holes. Ensure screws are screwed firmly to the wall.
6. When fitted, check the product is securely fixed to the wall and that the arm can be raised and lowered without interference, and that the leg is in contact with the floor when lowered. Also check that the friction lock is effective and that the support rail will hold itself at any angle during movement (i.e. it does not drop down on it's own).

5. CLEANING INSTRUCTIONS

Clean regularly using a general purpose, neutral detergent and warm water.

Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

DO NOT use solvents to clean the product (if painted).

To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water.

Allow product to fully dry prior to use/storage.

6. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

7. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: +44 (0)345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk