

General safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury. Retain these instructions in a safe place for future reference.

1.0 Intended Use

To enable children who cannot reach the floor when sat on a standard toilet, to sit comfortably with their feet supported. The frame provides support when getting on and off the toilet and the platform provides a sturdy base for them to step onto and rest their feet.

2.0 Safety Precautions

Always ensure the toilet platform is pushed up to the toilet as close as possible.

Do not exceed the safe working load of the product, 12 st 76 kg.

3.0 Unpacking

Remove packaging and check all parts carefully before assembling the product. Do not assemble if any parts are damaged.

Contents:

- Handle Frame X 2 (RH & LH)
- Platform
- Leg Extensions X 4
- 'E-clips' X 4

4.0 Assembly

Attach the platform to the LH & RH frames;

- Position the frames on either side of the platform such that the shortest end is at the front, figure 3.
- Depress the button clips on the frame stubs and slide the platform tubes over both the front and rear stubs, figure 4.

Ensure the button clip clicks back into place.

Maximum User Weight



Figure 1 – Toilet Platform



Figure 2 – Product Parts



Figure 3 – Handle frame position

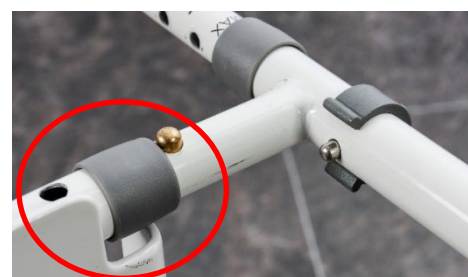


Figure 4 – Attaching the platform

Adjust the height:

- Remove the 'e-clips' and slide the legs to the desired position, figure 5.

Use the markings on the legs to ensure that all four legs are positioned at the same height to prevent rocking and instability.

- Align the holes and re-insert the 'e-clips', figure 6.

Place the frame around the toilet with the platform pushed back as close to the toilet as possible, figure 1.



Figure 5 – Positioning the leg extensions



Figure 6 – Securing the leg extensions

5.0 Cleaning

Clean regularly using a general purpose, neutral detergent and warm water.

Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.

To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.

Allow the product to dry thoroughly prior to use/storage.

6.0 Maintenance

Check that all fixing clips are inserted correctly and there is no excessive movement.

Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the mat and ferrules. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

7.0 Product Disposal

Product should be thoroughly cleaned prior to disposal.

Product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through local Trade Waste Collectors.

8.0 Product Guarantee

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS Healthcare Sherwood House, Cartwright Way, Forest Business Park

Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK

Tel. Customer Services: +44 (0)345 121 8111 | Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a Leading Supplier of Community Equipment and Daily Living Aids.

Please visit one of websites to view our full range of products:

www.healthcarepro.co.uk or www.completecareshop.co.uk