OTTER

Otter S

Otter M

Otter L

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Table of contents

Preface	8
Declaration of conformity	8
Service life	8
Disposal	8
Identification plate	8
Reuse.....	8
Warranty	8
Parts of the product	9
Intended purpose	9
Indication.....	9
Scope of delivery.....	9
Safety instructions.....	9
Assembly.....	10
Unfolding.....	10
Adjustment of the seat and backrest.....	10
Use of the headrest and strap	10
Optional accessories	10
Cover	10
Cleaning and maintenance	11
Technical data.....	11

Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co. KG declares compliance according to the medical device regulation (EU) 2017/745.

Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.


This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

Identification plate

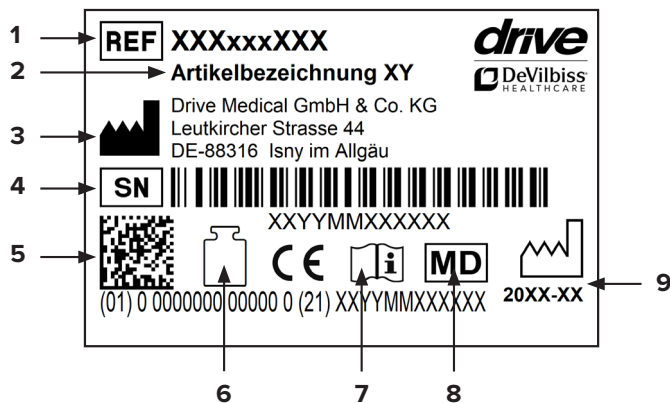1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year-month)

Reuse

This product can be reused. Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available from your specialist dealer or at www.drivedevilbiss.de

Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

EXCLUSION OF WARRANTY

Parts subject to wear include (among others):
Rubber feet

Parts of the product

1. Adjustable headrest
2. Adjustable chest strap
3. Frame
4. Adjustable support legs
5. Single-hand controls
6. Adjustable leg strap
7. Seat cover is water-permeable and washable

Intended purpose

Bathing supports are used in the bath. They are used particularly for children and young people e.g. with insufficient trunk control and/or hyperextension slant (e.g. in cases of cerebral palsy) from around the age of two. Bathing supports prevent or make it more difficult for the upper body to slip down and for the head to then go under the water.

Indication

INDICATION:

Lack of torso control, for example, in cases of paraplegia, central or peripheral neuromuscular diseases with weakness of the torso, paralysis of the lower extremities, tendency for hyperextension, for example, in the case of cerebral palsy. The products are used for children from approximately the age of two years onwards as well as adolescents and persons of small stature. The bath and shower support assists children and adolescents who cannot sit up freely and offers additional support by means of straps. For use in home care. Other use or use beyond this is considered to not be in compliance with intended use.

Scope of delivery

1 x	Child bath support
1 x	Headrest
1 x	Chest strap
2 x	Leg strap
1 x	Instructions for use

Safety instructions

- Read these instructions for use carefully.
- All components should be checked for damage and a secure fit prior to use. Do not use product if there are any defects!
- Caution – risk of slipping/falling!
- The product is significantly less stable than regular chairs, therefore avoid leaning to the side – risk of tipping!
- Always sit in the middle of the seat surface and all the way back against the backrest and not on the edge – risk of tipping and slipping!
- Take note of the permitted maximum weight of the user. This information can be found in the technical data in these instructions for use as well as on the identification plate.
- Never allow the child to sit in the bath and shower support unsupervised.
- Every serious incident which occurs in connection with

the product must be reported to the manufacturer and the competent authority of the member state in which the user and/or patient is located.


Assembly

The child bath support is preassembled (**photo 1**).

Unfolding

Pull the foot slightly out (**photo 2**) and pivot it until it stops (**photo 3**). Now press the foot backwards until it audibly clicks into place (**photo 4**). Check that it is locked properly.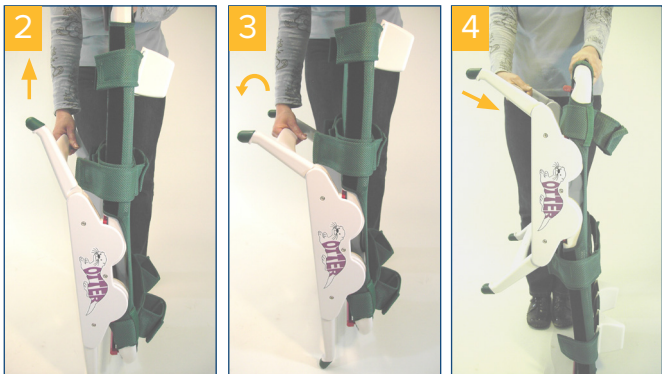

Adjustment of the seat and backrest

To adjust the seat surface or the backrest, pull the respective single-hand control (**photo 5-6**). Swivel the seat surface or backrest into the desired position (four increments possible). Ensure that the seat surface or backrest has correctly clicked into place.

Use of the headrest and strap

The chest strap and the headrest can be individually adjusted using a Velcro closure system (**photo 8**). The width of the headrest can additionally be adjusted by simply pushing both side supports.

Optional accessories


The following optional accessories are additionally available:

1. Shower support base frame
2. Bath support base frame

A click system enables quick conversions. Simply click the feet of the bath support into the red locking mechanisms. To disassemble, pull the red toggle lever.

Cover

To change the cover, remove the single-hand controls as shown (**photo 9**). Then the cover can be pulled off (**photo 10**). The cover can be washed in the machine up to 60°C.

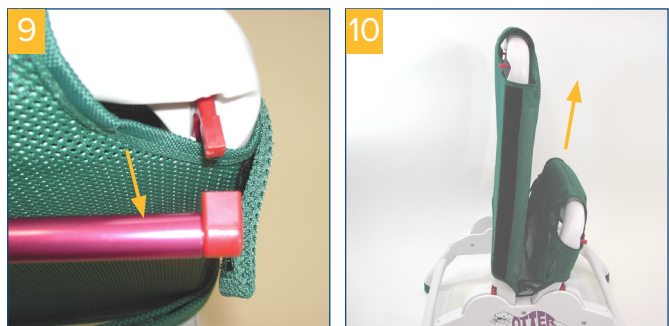Cleaning and maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary.
- The product should not be exposed to temperatures above 60°C for long periods of time (for example, use in a sauna, etc.)
- In the event of defects, contact your specialist dealer from whom you purchased this product.

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Drive Medical GmbH & Co. KG